

PRODUCT INFORMATION

Clinical Center Standardization Committee




**ABBOTT AIM PLUS
AMBULATORY PUMP TUBING CHANGE
NOVEMBER 2003 (01)**

DISCONTINUED—Abbott Set #13460

The manufacturer has discontinued the latex-free tubing 'Pump Set-SL with Quick Load Cartridge, Add-on Anti-Siphon Valve and Female Luer Lock Adapter, 13 Inch'.

REPLACEMENT—Abbott Set #13732

Latex-free tubing 'Pump Set with Injector, Quick Load Cartridge, 1.2 Micron Filter, Luer Lock Adapters, Integral Anti-Valve, Nonvented, 72 inch'.

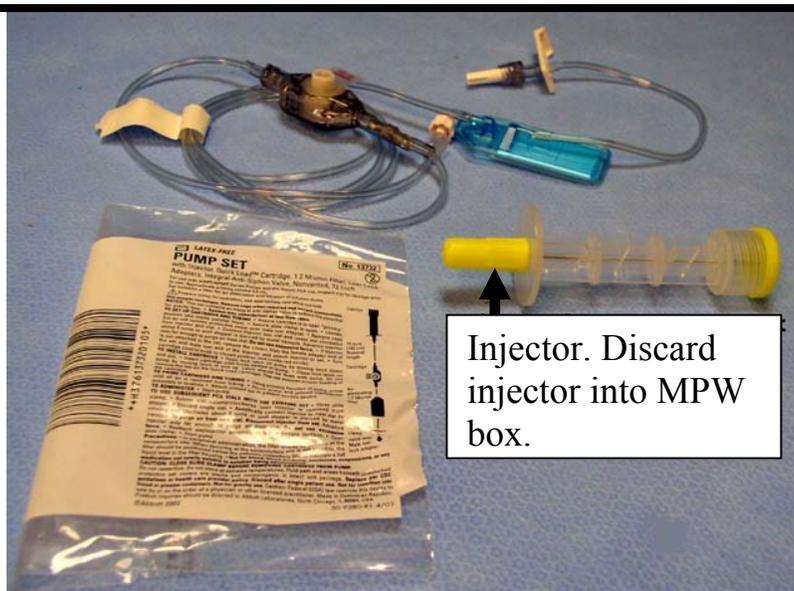

Item #: 00625 Unit of Issue: Each

**How to find in VSC: Cartridge, Abbott,
Cartridge Infusion Provid, 00625**

Instructions for Use

- ➔ Throw away the injector.
- ➔ The dead space is 6 ml (5.6ml priming volume). Set the pump to indicate a fill volume of 44ml for a 50 ml bag (94ml for a 100ml bag) or the patient won't get anything near the end.
- ➔ There is a disclaimer in fine print on the bag that says it may "underdeliver at low flow rates". Be careful to monitor pump.

Questions: Contact Mike Kolf, Pharmacy, email or x66551.